Pak-EDGE Packager Procedures

When adding a new drug to the library, add in the following format:

NDC = How the NDC appears on the bottle


Drug ID = NDC # in the 5, 4, 3 format with dashes

Line 1 = Generic Name

Line 2 = Brand Name, if the drug is the generic you need to add

USE FOR in front of the Brand Name. If a Generic drug without a Brand name, list Generic Name on line 1 only. DO NOT REPEAT on line 2.


Line 3 = Strength

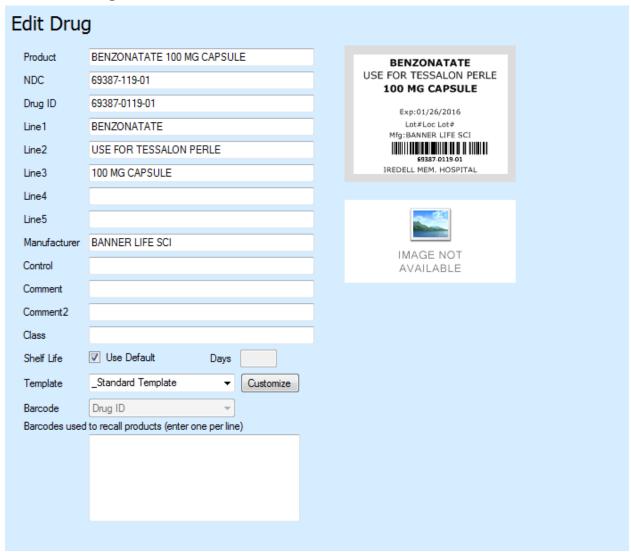

Notice the template is _Standard Template .

Example of a control drug from a bulk bottle built in the library.

Notice the template is Control Template which adds the CV to the package.

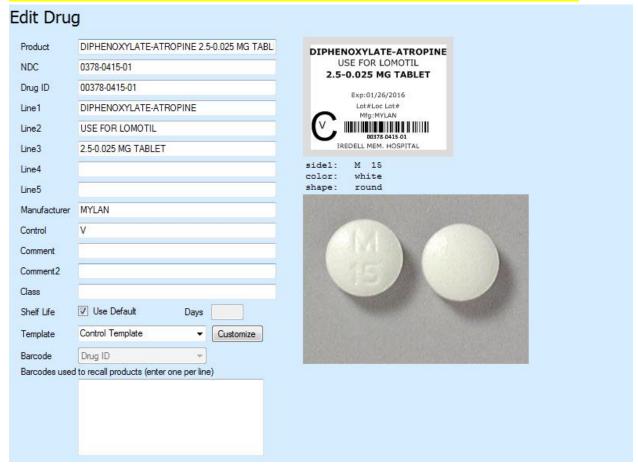

Flag Labels from Ricky

When adding a new drug to the library, add in the following format:


NDC = How the NDC appears on the bottle

Drug ID = NDC # in the 5, 4, 3 format without dashes

Line 1 = Generic Name

Line 2 = Brand Name, if the drug is the generic you need to add USE FOR in front of the Brand Name. If a Generic drug without a Brand

Line 3 = Strength

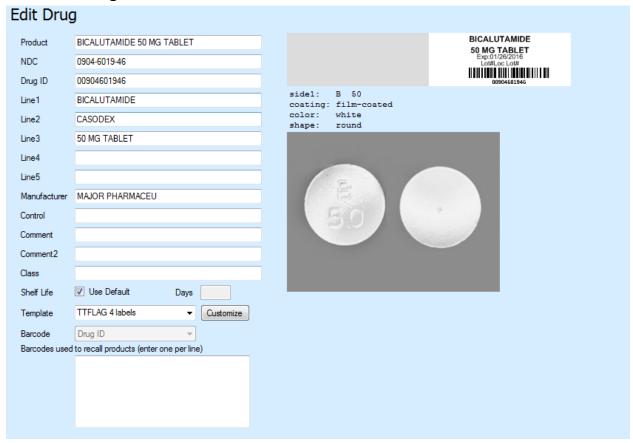


This will be our most used label.

Notice the template used is the TTFLAG 4 labels which are the tadpole labels we used to use with the McKesson Printer.

Flag Labels from Ricky

Here is an example of a Control Drug that is built in the library.


Notice the template selected is TTFLAG 4 label Control which adds the C2 to the label.

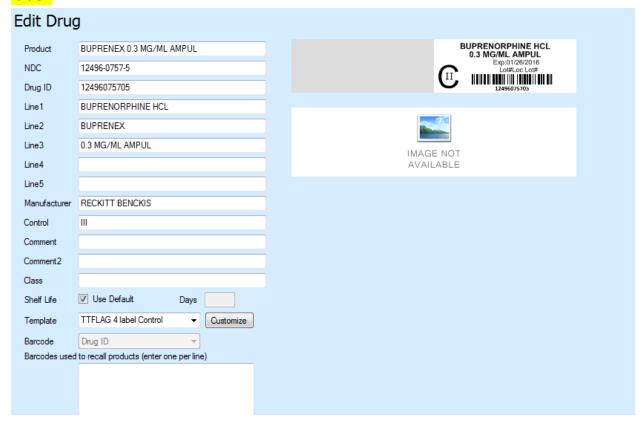

Flag Labels from Ricky


This is an example of an oral liquid that will be pulled up in a syringe. This label should **only** be used for labeling syringes.

Notice the template is Clear Flag 3.5in x 1 in

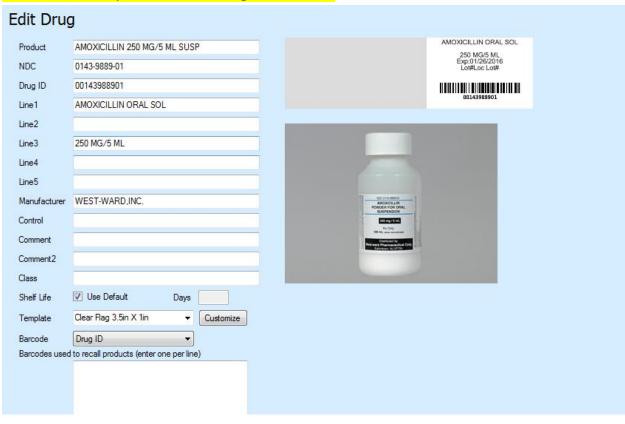

Labels from FRED

TO GO PACK Labels should all be built in the library. However, if you need to build one you can go to the library and copy an existing item.

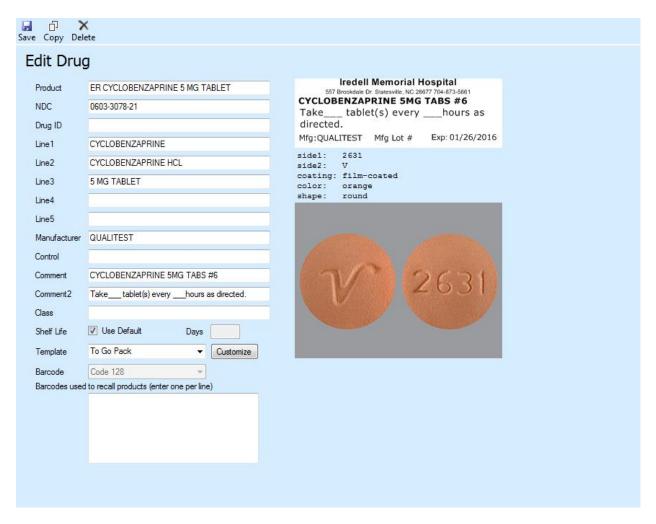


Select the Copy function above Edit Drug. You will need to rename your label to match the drug you are adding. Then replace the appropriate fields to match your medication, and then save your new label.

Notice the template is TO GO PACK.

Labels from FRED

This is another example of the white label from Fred used for To go packs.

Notice the template is Plain Label.

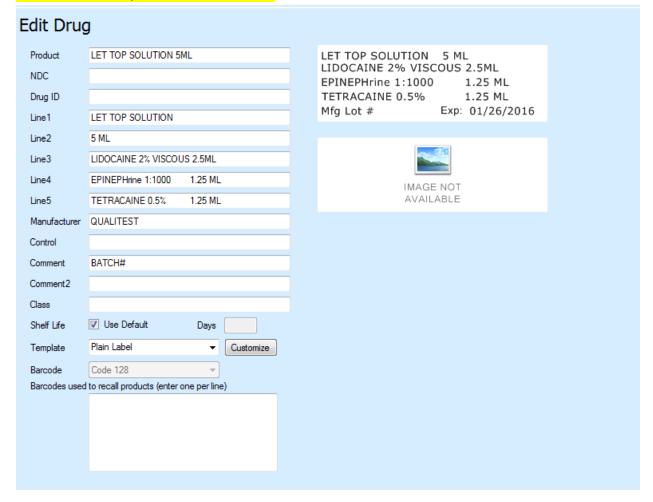

Important Information for packager:

- 1. Use the templates that are listed in this memo. These have the margins set up and already and have been tested to make sure they print correctly.
- 2. If the DRUG ID is blank the barcode will not print on the label.
- 3. If you are changing a medication from plain to ER or from ER to plain please double check with Diane or Amy so we can compare Amerisource to see what was ordered verses what we received in.
- 4. If you are approving a label and you have to change the information that was put in please go over with the person that added it to library for education purposes.
- 5. Make sure the drug information fits on the label and it's not running off the sides.
- 6. If you have questions on how the drug should be set up please check with Amy as she can pull up our old packaging system to see how it was set up in that software.
- 7. Please remember to post verify the ER to go packs in the packaging software.